



Patient Reported Satisfaction: 3-Year Multicenter Results

PURPOSE

This sensor-assisted, multicenter study cohort was prospectively assessed for concluding (3-year) patient-reported outcomes. Scores were compared to existing peer-reviewed publications reporting a similar follow-up interval (2-3 years) to determine the impact of consistent TKA soft tissue balance on patient satisfaction.

METHODS

- 129 sensor-assisted patients (quantifiably balanced)
- 7-question survey administered (5-point Likert scale)
- Literature review conducted via PRISMA guidelines


RESULTS

- 98.3% of balanced, sensor-assisted patients reported being “satisfied” to “very satisfied” at 3-years post-op¹
- On average, 87% of patients reported being “satisfied” to “very satisfied” in comparative literature during the same follow-up interval²⁻⁹


CONCLUSION

THE VERASENSE GROUP EXHIBITED THE HIGHEST REPORTED SATISFACTION AMONG CONTEMPORARY TECHNOLOGIC AND MANUAL SURGICAL MODALITIES FOR THE SAME 3-YEAR POST-OP INTERVAL.

2-3 YEAR PATIENT REPORTED SATISFACTION: COMPARISON OF LITERATURE

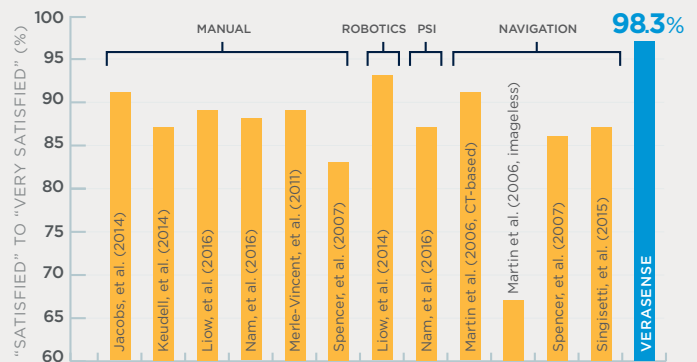

ITEMIZED SATISFACTION QUESTION SCORES

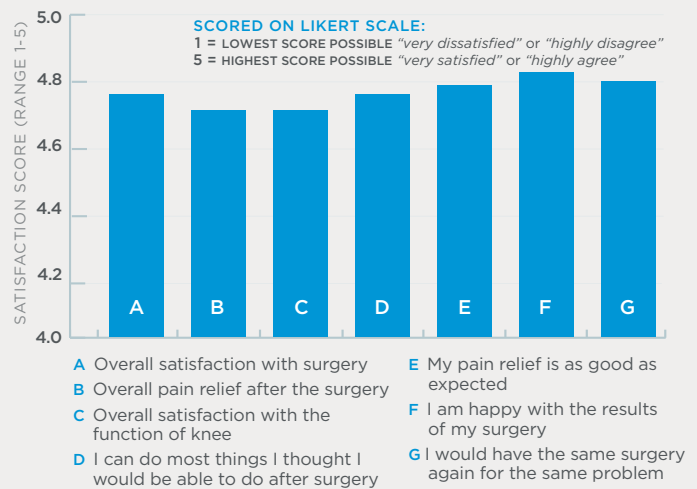

*Dr. Martin Roche participated in this research and is Founder and Chief Medical Officer of OrthoSensor, Inc.

1. 2-Year, 3-Year Clinical Outcomes, OrthoSensor Multicenter Evaluation. Pending Publication. Data on file at OrthoSensor, Inc.
2. Keudell AV, Sodha S, Collins J, Minas T, Fitz W, Gomoll AH. Patient satisfaction after primary total and unicompartmental knee arthroplasty: an age-dependent analysis. *Knee*. 2014; 21: 180-184.
3. Jacobs CA, Christensen CP, Karthikeyan T. Patient and intraoperative factoris influencing satisfaction two to five years after primary total knee arthroplasty. *J Arthroplasty*. 2014; 29: 1576-1579.
4. Nam D, Nunley RM, Berend KR, Lombardi AV, Barrack RL. The impact of custom cutting guides on patient satisfaction and residual symptoms following total knee arthroplasty. *Knee*. 2016; 23: 144-148.
5. Merle-Vincent F, Couris CM, Schott AM, Conrozier T, Piperno M, Mathieu P, Vignon E. / factors predicting patient satisfaction 2 years after total knee arthroplasty for osteoarthritis *Joint Bone Spine*. 2011; 78: 383-386.
6. Spencer JM, Chauhan SK, Sloan K, Taylor A, Beaver RJ. Computer navigated versus conventional knee replacement. *Bone Joint J*. 2007; 89-B: 477-480.
7. Singiseti K, Muthumayandi K, Abual-Rub Z, Weir D. Navigation-assisted versus conventional total knee replacement: no difference in patient-reported outcomes measures (PROMs) at 1 and 2 years. *Arch Orthop Trauma Surg*. 2015; 135:1595-1601.
8. Liow MHL, Goh GSH, Wong MK, Chin PL, Tay DKJ, Y SJ. Robotic-assisted total knee arthroplasty may lead to improvement in quality-of-life measures: a 2-year follow-up of a prospective randomized trial. *Knee Surg Sports Traumatol Arthrosc*. 2017 Sep;25(9):2942-2951.
9. Martin A, von Stempel A. Two-year outcomes of computer tomography-based and computed tomography free navigation for total knee arthroplasties. *Clin Orthop Relat Res*. 2006; 49: 275-282.